

Restore Foam Dressing with Adhesive



Protocol

This protocol is intended to supplement, not replace, your facility's guidelines and/or common practices.

Desired Outcomes

- To maintain a moist wound environment for the healing of partial to full-thickness wounds
- To prevent pain during dressing removal
- To absorb exudate

Equipment

1. Restore Foam Dressing with Adhesive (Choose a size which ensures that the central pad will cover the entire wound)
2. Wound measuring guide
3. Restore Wound Cleanser or sterile saline for wound cleansing
4. Gauze or towel to dry periwound skin
5. Secondary cover dressing
6. Gloves

Procedures

- Preparation of wound area
- Application of wound dressing
- Removal of wound dressing
- Documentation of observations

Preparation of Wound Area

1. Assemble supplies and take to bedside.
2. Identify and inform patient of procedure.
3. Provide for privacy.
4. Wash hands and apply gloves.
5. Position patient so that the wound area is exposed.
6. Thoroughly rinse or irrigate the wound area with Restore Wound Cleanser or sterile saline, if appropriate. *(If necessary, the wound should be debrided).*
7. Clean and dry the periwound skin.
8. Measure the wound using the wound measuring guide.

Application of Dressing

1. Remove clear plastic tabs from the contact layer of the Restore Foam Dressing with Adhesive
2. Apply Restore Foam Dressing with Adhesive to cover the wound using aseptic technique. The foam pad should cover the entire wound.
3. Gently press down on the adhesive to secure.
4. Remove gloves and wash hands after completing procedure.

Removal of Dressing

1. Wash hands and put on gloves.
2. Remove Restore Foam Dressing with Adhesive
3. Irrigate wound base using Restore Wound Cleanser or sterile saline.
4. Reapply dressing if necessary.
5. Remove gloves and wash hands after completing procedure.

Note: Restore Foam Dressing with Adhesive may be left in place for up to four days. Dressing change frequency will depend on patient condition and the level of exudate.

Documentation of Observations

Document dressing changes, wound measurements and other pertinent observations regarding the wound or periwound site.

Assess

Key observations to document with wound dressing change:

- Size of wound
- Depth of tissue involvement (partial-thickness, full-thickness, stage)
- Presence of undermining or tunnelling
- Anatomic location
- Wound Base (granulation, epithelialization, subcutaneous tissue, muscle, bone or other underlying structures, non-viable/necrotic tissue, color, exudate amount/odor/color)
- Edge of wound
- Presence of foreign bodies
- Condition of surrounding skin

Plan

Select interventions to provide appropriate local wound care:

- Prevent and manage infection
- Pain management
- Cleanse wound
- Remove nonviable tissue
- Manage exudate
- Eliminate dead space
- Control Odor
- Protect wound and periwound skin
- Maintain a moist wound environment

Implement

- Follow Protocol for the Restore Foam Dressing with Adhesive
- Reduce or eliminate factors that may interfere with wound healing
- Provide systemic support for wound healing

Evaluate

- Reassess wound with each dressing change.
- Frequency of dressing changes should be dictated by facility protocol.
- If no progress can be demonstrated within two to four weeks, reevaluate the overall treatment plan, as well as adherence to this plan, making modifications as necessary.

Restore Foam Dressing with Adhesive Advantages

- Maintenance of a moist wound environment
- Minimizes or eliminates trauma to the wound bed upon dressing removal
- Helps reduce or eliminate pain at dressing changes
- Super absorbent foam pad provides vertical absorption of wound exudate and helps protect the surrounding skin from maceration
- Suitable for use under compression bandages
- Does not require a secondary dressing

See Instructions for Use for important information regarding the use of this product at www.hollisterwoundcare.com/products/ifus.html.

Ordering Information

Restore Foam Dressing with Adhesive

	Stock No	Unit of Sale
4" x 4", Sterile (10 cm x 10 cm) with 1.5" x 1.5" (3.8 cm x 3.8 cm) foam pad	509347	10

6" x 6", Sterile (15 cm x 15 cm) with 4" x 4" (10 cm x 10 cm) foam pad	509348	10
--	--------	----

6" x 8", Sterile (15 cm x 20 cm) with 3.5" x 5.6" (8.9 cm x 14.2 cm) foam pad	509349	10
---	--------	----

Restore Wound Cleanser

8 oz. (236 mL) Trigger Spray Bottle, Non-Sterile	529975	12/box
--	--------	--------

12 oz. (354 mL) Trigger Spray Bottle, Non-Sterile	529976	12/box
---	--------	--------



An alliance of Hollister Incorporated and Laboratoires URGO

Manufactured for
Hollister Wound Care LLC
1580 South Milwaukee Avenue
Suite 405
Libertyville, Illinois 60048
1.888.740.8999

Distributed in Canada by
Hollister Limited
95 Mary Street
Aurora, Ontario L4G 1G3
1.800.263.7400

www.hollisterwoundcare.com