

Restore DUO Absorbent Dressing with TRIACT Technology



Protocol

This protocol is intended to supplement, not replace, your facility's guidelines and/or common wound care practices.

Desired Outcomes

- To maintain a moist wound environment for the healing of partial thickness wounds
- To manage low exuding acute and chronic wounds
- To help prevent pain during dressing removal

Equipment

1. Restore DUO Absorbent Dressing.
2. Wound measurement guide.
3. Restore Wound Cleanser or other solution, as indicated, for wound cleansing.
4. Gauze or other material to dry periwound skin.
5. Secondary cover dressing to secure dressing in place.
6. Gloves.

Procedures

- Preparation of wound area
- Application of dressing
- Removal of dressing
- Documentation of observations

Preparation of Wound Area

1. Assemble supplies and take to bedside.
2. Identify and inform patient of procedure.
3. Provide for privacy.
4. Wash hands and apply gloves.
5. Remove dressing if present.
6. Thoroughly rinse or irrigate the wound area with Restore Wound Cleanser or sterile saline, if appropriate. *(If necessary, the wound should be debrided).*
7. Clean and dry the periwound skin.
8. Measure the wound using the wound measuring guide.
9. Choose a dressing size which ensures the pad will cover the entire wound.

Application of Dressing

1. Remove the dressing from the package. Remove the protective tabs from the dressing.
2. Apply the dressing directly to the wound with the TRIACT mesh toward the wound.
3. Cover with secondary dressing and secure with tape or other appropriate material.
4. Remove gloves and wash hands after completing procedure.

Removal of Dressing

1. Wash hands and put on gloves.
2. Remove secondary dressing and the Restore DUO dressing and dispose of properly.
3. Cleanse the wound bed following your facility's protocol.
4. Reapply dressing if necessary.
5. Remove gloves and wash hands after completing procedure.

Note: The frequency of dressing changes for Restore DUO Absorbent dressing depends on the clinical presentation of the wound (e.g. amount of exudate and healing progression). The dressing may be left in place for a maximum of seven days.

Documentation of Observations

Document dressing changes, wound measurements and other pertinent observations regarding the wound or periwound site.

Assess

Key observations to document with wound dressing change:

- Size of wound
- Depth of tissue involvement (partial-thickness, full-thickness, stage)
- Anatomic location
- Wound base (granulation, epithelialization, nonviable tissue, color, amount of exudate, odor etc.)
- Edge of wound
- Presence of foreign bodies (e.g. suture)
- Condition of periwound skin

Plan

Select interventions to provide appropriate local wound care:

- Prevent and manage infection
- Cleanse wound
- Remove non-viable tissue
- Control odor
- Eliminate dead space
- Protect wound and periwound skin
- Maintain a moist wound surface

Implement

Follow Protocol for Restore DUO Absorbent Dressings

- Reduce or eliminate factors that may impede wound healing
- Provide systemic support for wound healing (proper nutrition, etc.)

Evaluate

- Reassess wound with each dressing change
- Frequency of dressing changes should be dictated by facility protocol
- Discontinue Restore DUO Dressing if wound develops moderate amount of exudate
- If no progress can be demonstrated within two to four weeks, reevaluate the overall treatment plan, as well as adherence to this plan, making modifications as necessary.

Ordering Information

Restore DUO Absorbent Dressing with TRIACT Technology

	Stock No	Qty per Box
2" x 4" (5cm x 10cm)	509375	10
4" x 5" (10cm x 12.7cm)	509376	10
6" x 8" (15cm x 20cm)	509377	10

Restore Wound Cleanser

8 oz. (236 mL) pump spray	529975	12
12 oz. (354 mL) pump spray	529976	12

See Instructions for Use for important information regarding the use of this product at www.hollisterwoundcare.com/products/ifus.html.

Contraindications

Restore DUO Absorbent Dressing should not be used on individuals who are sensitive to or who have had an allergic reaction to the dressing or one of its components.



Manufactured for
Hollister Wound Care LLC
1580 South Milwaukee Avenue
Suite 405
Libertyville, Illinois 60048
1.888.740.8999

Distributed in Canada by
Hollister Limited
95 Mary Street
Aurora, Ontario L4G 1G3
1.800.263.7400